

MYCOPLASMA ID/AST TEST KIT

Used for:
Mycoplasma Uu
Mh infection identification
Sensitivity testing

inovaturk



Features:

SCAN-10 Auto sampling instrument

- 1. Rapid sampling with standard operation
- 2. English interface. Easy to use



+90 (212) 210 20 20



info@inovaturk.com.tr



www.inovaturk.com.tr

Ataşehir bulvarı, Atatürk Mah. Evinpark Rezidans No:23 a-1 blok d.24 Ataşehir / İSTANBUL

MYCOPLASMA ID/AST TEST KIT

Package: 20 tests/KIT

Storage: 2-8°C

inovaturk



REF 270102003

Drugs name	Abbr.	Concentration (mg/L)	
Lincomycin	LIN	8	1
Ofloxacin	OFL	4	1
Erythromycin	ERY	4	1
Ciprofloxacin	CIP	2	1
Roxithromycin	ROX	4	1
Sparfloxacin	SPA	4	1
Azithromycin	AZI	4	1
Levofloxacin	LEV	4	1
Josamycin	JOS	8	2
Clarithromycin	CLA	4	1
Doxycycline	DOX	8	4
Clindamycin	CLI	4	1

Features

- Easy to operate
- High nutrient broth ensures rapid growth
- Much visible color for reading
- Identification and sensitivity combo test

Principle

- Colorimetry for identification and sensitivity test
- Used for Ureaplasma urealyticum(Uu) and Mycoplasma hominis(Mh) sample
- If over or equal to 10^4 CCU/ml, it will be counted as infection

Procedure:



Mix reagent and add in C-well



Inoculate swab



Sampling



Incubation for 1-2 days



Reading and report